

How to use

A. Preparation before use

- 1) Read the user manual thoroughly before use.
- 2) Do not use the product if the packaging is damaged.
- 3) Check the expiration date of the product and do not use it if it has expired.
- 4) Check if any of the product components are missing.
- 5) Use at a temperature of 5~40°C, humidity of 20~75%, and air pressure of 70~106kPa.

B. Operating Instructions

- 1) Remove the product from the packaging.
- 2) Close the clip.
- 3) Connect the insertion tip of the saline supply line to the irrigation solution (saline) bag.
- 4) Connect the spray tip to the endoscope or place it on the surgical site (or wound).
- 5) Press the power button (🔌) to turn on the power, and the top LED lights up.

(At this point, pressing the ENTER button (↩) will start the action.)



- 6) If you need to adjust the pressure, do the following
 - (1) Press the mode button (☰) once to select the pressure setting mode.
 - (2) Press the pressure control buttons (⬆) (⬇) to set the desired pressure.
 - (3) Press mode button (☰) to save the set pressure value.
- 7) Open the clip.
- 8) Press the ENTER button (↩) again to start blowing at the set pressure value.
- 9) When you're done, close the clip and press the power button to turn it off.

C. Storage and care after use

- 1) Dispose of the battery in this product as regular waste.
- 2) Dispose of the remaining parts except the battery in accordance with medical waste regulations.
- 3) This product is a disposable medical device and should not be resterilized or reused.